

**ONLY**  
**90**  
**SECONDS**  
*in the Mouth*

## Fast and Accurate Impressions



Load the tray with Exafast Heavy Body or Monophase. Easy to extrude cartridge and dispenser let you fill a tray quickly and easily. (New Exafast Putty is also available.)



Have patient occlude and/or hold impression in position for 90 seconds. (When using putty, have patient occlude for 2 minutes and 15 seconds.)



Extrude Exafast Injection or Regular Type directly on to the prepared teeth. Material stays in place until occlusion. Monophase material can also be loaded into a syringe and applied directly to the prepared area.



Remove finished impression from the patient's mouth; rinse and disinfect.



Place the loaded tray over the prepared area and seat in the mouth.



Exafast impressions, just like Exaflex and Examix impressions, can be easily disinfected by immersing in an alkaline glutaraldehyde (such as COECIDE™ XL or COECIDE XL Plus) or spraying with a hard surface disinfectant (GC Spraycide™). Follow instructions provided by manufacturer. Just rinse with water prior to and after disinfecting.

# EXAFAST™

Fast Setting VPS Impression Material



### Product Code

**Exafast NDS Two-Cartridge Package (48ml) with Tips**  
Contains: Two 48ml cartridges (96ml total), 6 mixing tips and instructions.  
**137107** EXAFAST NDS Injection  
**137007** EXAFAST NDS Regular Type  
**137407** EXAFAST NDS Monophase  
**137207** EXAFAST NDS Heavy Body

**Exafast NDS Two-Cartridge Package (75ml) with Tips**  
Contains: Two 75ml cartridges (150ml total), 9 mixing tips and instructions.  
**137275** EXAFAST NDS Heavy Body

**137295** **Exafast NDS Super Value Heavy Body Pack**  
Contains: Twenty 75ml cartridges (1500ml total), 90 mixing tips and cartridge dispenser 2 NDS 75ml

**137307** **Exafast Putty Standard Package**  
Contains: One jar putty base and one jar putty catalyst (44gm each), instructions and scoops.

**137320** **Exafast Putty Clinic Package**  
Contains: Five jars putty base and five jars putty catalyst (44gm each), instructions and scoops.

### Accessories


**153148** Mixing Tips NDS, Type 2S (pink), 48 tips  
**153248** Mixing Tips NDS, Type 2L (aqua), 48 tips  
**153048** Mixing Tips NDS, Type 2LL (purple), 48 tips  
**150100** Intraoral Tips NDS (for pink or aqua Mixing Tips), 100 tips  
**152101** GC Cartridge Dispenser 2 NDS 48ml  
**152102** GC Cartridge Dispenser 2 NDS 75ml  
**152171** Plunger Conversion Kit for 75ml cartridge, plunger and instructions  
**138906** VPS Tray Adhesive, pkg. of six 7ml bottles  
**138991** VPS Tray Adhesive, 7ml bottle  
**250116** GC Tray Adhesive Remover, 16 oz. bottle  
**159001** COE Aluminum Syringe, complete  
**159225** Syringe Tips, for stainless steel and aluminum syringes, Type A (Slow Curve), pkg. of 25  
**159302** GC Plastic Syringe Kit, complete  
**159300** GC Plastic Syringe Tips, pkg. of 100  
**159310** GC Plastic Syringe Tips, pkg. of 25

# EXAFAST™

Fast Setting VPS Impression Material



## EXA Accurate EXA Fast

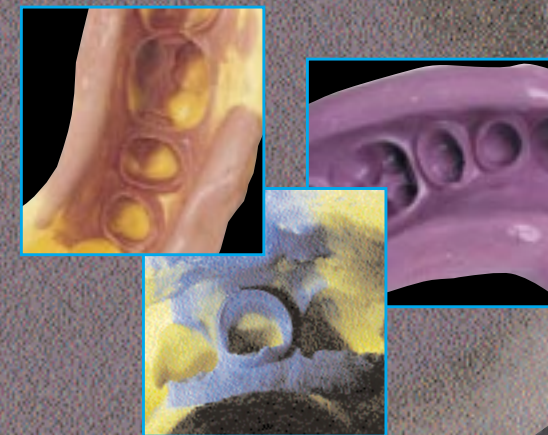

GC America Inc.

Advancing the Art and Science of Dentistry

© 2000 GC America Inc., 3737 West 127th St., Alsip, Illinois 60803

<http://www.gcamerica.com>  
Lit. Code 813153

# EXAFAST™

## the next generation



### NEW EXAFAST lets you produce extraordinarily smooth and precise impressions in 90 seconds...easily.


EXAFAST is a fast setting vinyl polysiloxane impression material developed to be consistently accurate, increase productivity and maximize patient comfort.

- The proven quality and performance of EXAFLEX and EXAMIX.
- Features the New Easy to Extrude Delivery System - NDS.
- More accurate 2-1/4 minute EXAFAST Putty.
- New Super Value 75ml Heavy Body Cartridge .

### EXA Fast

Less time in the mouth minimizes the chances for distortion due to patient movement, prevents patient stress and reduces chair time. EXAFAST is a vast productivity improvement for the dentist and staff and it maximizes patient comfort without compromising accuracy.

- Cartridge material has 60 seconds working time; sets in 90 seconds in the mouth.
- Putty has 45 seconds working time; sets in 2-1/4 minutes in the mouth.

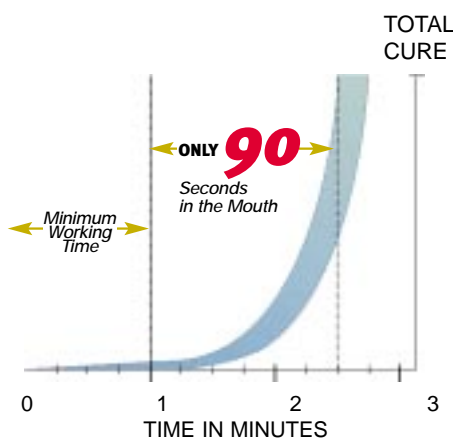

### EXA Handling

Just like EXAMIX and EXAFLEX, EXAFAST flows easily around preps to capture details and eliminate voids - even in moisture compromised situations.

#### Thixotropic

- Ideal consistency (stacks nicely) and flow under pressure make EXAFAST easy to handle and extremely accurate. It penetrates critical, hard to reach areas.
- INJECTION stays in place on the teeth until patient occludes.

#### Hydrophilic

- Uniformly flows over moist surfaces and oral tissue to reproduce complete and accurate details.
- Makes it easy to pour accurate gypsum models.

#### Non-Slumping Heavy Body

- Provides excellent seating compression.
- Forces injection material into all areas of the preparation.



### EXA Accuracy

EXAFAST lets you accurately replicate all the details. Produce highly precise impressions - EXAFAST.

#### Unsurpassed Dimensional Stability

- Won't shrink or become distorted (less than 0.2% shrinkage @ 24 hours).
- Pour multiple, consistently accurate models from a single impression.
- Accurate model can be poured immediately or up to two weeks after making the impression.

#### Hydrogen Scavenger

- Allows immediate pour of model by eliminating the chance of porosity from escaping gas bubbles.

#### Excellent Elastic Recovery and Tear Strength

- Impressions can be removed from prepared areas, even from undercuts, without tearing or distortion.
- Models can be removed from the impression without breakage or deformation.

### Now in 5 Viscosities to Fit Any Need

- INJECTION - Ideal consistency and flow under pressure for the perfect wash material.
- REGULAR TYPE - Perfect for those who like their wash material a bit more thixotropic and a little less runny.
- MONOPHASE - Syringeable, yet enough compression to attain the accuracy you want from a VPS impression.
- HEAVY BODY - The "high compression" tray material.
- PUTTY - The NEW fast set version of GC's famous EXAFLEX Putty.

- EXAFAST NDS INJECTION Syringe/Wash
- EXAFAST NDS REGULAR Syringe/Wash
- EXAFAST NDS MONOPHASE Tray/Syringe
- EXAFAST NDS HEAVY BODY Tray
- EXAFAST PUTTY Tray

### EXA Easy to Use

Designed for simplicity.

#### New Delivery System NDS

- New Easy-to-Extrude Cartridge Dispenser 2.
- New cartridge prevents contamination and tip clog.



#### Physical Properties

Working Time	1 minute (cartridge)	
	45 seconds (putty)	
Setting Time	90 seconds in mouth (cartridge)	
	2-1/4 minutes in mouth (putty)	
Recovery from Deformation	99.7% INJ 99.3% HB	99.5% REG 99.3% MONO 99.2% PUTTY
Max. Strain in Compression	≤7.0% INJ ≤3.3% HB	≤4.8% REG ≤3.2% MONO ≤1.2% PUTTY
Linear Dimensional Change after 24 hrs.	≤0.2%	

Test conditions: Temperature 23° (±2°C), Relative Humidity 50 (±5%)  
ISO 4823: 1992 (E)



**Don't settle for just fast...Go for EXAFAST, the fast impression material with proven EXA quality and performance!**